RespiChamber*

HOSPITAL

AUTOCLAVABLE VALVED HOLDING CHAMBER

For Use With Metered Dose Inhalers

Made in Canada by
Trudell Medical International
725 Third Street, London ON, Canada N5V 5G4
☎ +1-519-455-7060
www.respichamber.com

Trudell Medical International Europe Limited
BioCity Nottingham, Pennyfoot Street,
Nottingham, United Kingdom NG1 1GF
www.trudellmed.com

ENGLISH

Ensure that validated procedures are used for cleaning, disinfection and sterilization, and that the validated parameters are observed in each cycle.

CLEANING AND DISINFECTION

1. Remove the backpiece and the mask from the chamber.
2. Soak the parts in a 0.5% solution of Mediclean Forte (or similar) cleaning agent for 5 minutes agitating gently.
3. Place the parts (backpiece, chamber and mask) in the washer/disinfector ensuring the chamber is upright.
4. Select the cleaning program and follow manufacturer's instructions. Ensure a pH neutral cleaning agent is used.
5. Following disinfection, remove components from washer/disinfector and shake to remove excess water. Allow to air dry thoroughly prior to continuing to sterilization.

The effectiveness of this procedure has been proven compliant to EN ISO 14937:2009 using the Vario TD cleaning cycle in a 7836CD Washer/Disinfector made by Miele, using the cleaning agent Neodisher[†] Mediclean[†] (Dr. Weigert[†], Hamburg, Germany).

STERILIZATION

1. After cleaning and disinfection, pack the dismantled chamber in a sterilization pouch.
2. Autoclave the **RespiChamber**[®] HOSPITAL chamber according to the autoclave manufacturer's instructions:
 - 134°C (to be maintained for 3.5 minutes)

Ensure the chamber components do not come in contact with other materials, metal trays or shelving in the autoclave and that nothing is placed on top of the device during reprocessing.
3. Re-attach the backpiece (and mask if applicable) to the chamber prior to next use.

⚠ CAUTIONS

- Clean, disinfect and sterilize before every change in patient.
- Use of cleaning and disinfection procedures other than those listed above have not been validated and may cause damage to the device.
- Do not disassemble the product beyond what is recommended in the cleaning instructions or damage may result.

Description	Re-Order Number
RespiChamber [®] HOSPITAL Autoclavable Valved Holding Chamber	11350591120
Small Mask	11353691050
Medium Mask	11353791050
Large Mask	11355791050

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REF 113505

113024-001 Rev 01 * trade marks and registered trade marks of Trudell Medical International. † trade marks and registered trade marks of the respective companies. Copyright © Trudell Medical International 2015. All rights reserved.

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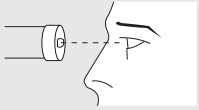
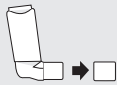
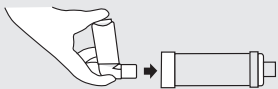
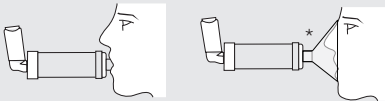
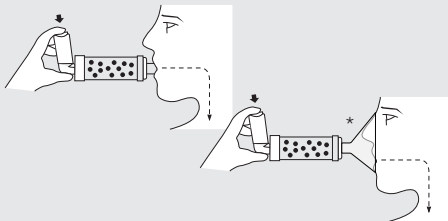
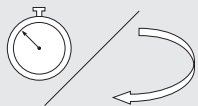
INSTRUCTIONS FOR USE

INTENDED USE

The **RespiChamber*** HOSPITAL Autoclavable Valved Holding Chamber is designed for multi-patient use in a hospital environment under the supervision of a healthcare professional. The chamber is intended to be used along with a metered dose inhaler to deliver aerosol medication to the patient's lungs.

Before use, ensure these instructions and the instructions supplied with the inhaler have been read and are kept available at all times.

Before use, carefully examine the chamber. Replace immediately if any defect is noticed.

Remove cap from inhaler.

Shake the inhaler immediately before each use as per the instructions supplied with it.

Insert the inhaler into the backpiece of the chamber.

MOUTHPIECE: Put mouthpiece into mouth and ensure an effective seal.

MASK: If applicable, slide mask onto the mouthpiece of the chamber. Align mask to inhaler per picture. Apply mask to face and ensure an effective seal.

MOUTHPIECE: Press the inhaler at the beginning of a slow inhalation. The patient should inhale slowly and deeply through the mouth until a full breath has been taken followed by a 5-10 second breath hold, if possible. Alternatively, the patient can keep their mouth tight on the mouthpiece and breathe slowly 2-3 times through the chamber after the inhaler is pressed.

MASK: Press the inhaler at the beginning of a slow inhalation. Maintain seal for 5-6 breaths after the inhaler is pressed.

Follow instructions supplied with the inhaler on how long to wait before repeating steps 3-6 as prescribed. Administer one puff at a time.

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