# RespiChamber<sup>®</sup>

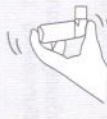
HOSPITAL  
AUTOCLAVABLE VALVED HOLDING CHAMBER

1


2


3

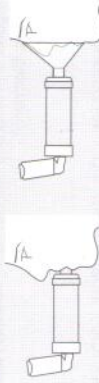4

5

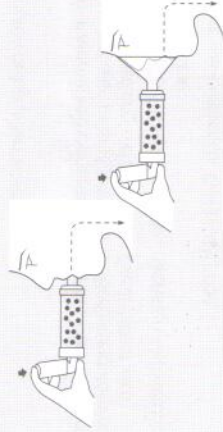6

7

## ENGLISH

### INSTRUCTIONS FOR USE

#### INTENDED USE

The **RespiChamber<sup>®</sup>** HOSPITAL Auto-clavable Valved Holding Chamber is designed for multi-patient use in a hospital environment under the supervision of a healthcare professional. The chamber is intended to be used along with a metered dose inhaler to deliver aerosol medication to the patient's lungs. Before use, ensure these instructions and the instructions supplied with the inhaler have been read and are kept available at all times.

Before use, carefully examine the chamber. Replace immediately if any defect is noticed.

Remove cap from inhaler.

Shake the inhaler immediately before each use as per the instructions supplied with it.

Insert the inhaler into the backpiece of the chamber.

**MOUTHPIECE:** Put mouthpiece into mouth and ensure an effective seal.

**MASK:** If applicable, slide mask onto the mouthpiece of the chamber. Align mask to inhaler per picture. Apply mask to face and ensure an effective seal.

**MOUTHPIECE:** Press the inhaler at the beginning of a slow inhalation. The patient should inhale slowly and deeply through the mouth until a full breath has been taken followed by a 5-10 second breath hold, if possible. Alternatively, the patient can keep their mouth tight on the mouthpiece and breathe slowly 2-3 times through the chamber after the inhaler is pressed.

**MASK:** Press the inhaler at the beginning of a slow inhalation. Maintain seal for 5-6 breaths after the inhaler is pressed.

Follow instructions supplied with the inhaler on how long to wait before repeating steps 3-6 as prescribed. Administer one puff at a time.

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